

Midwest® Lowspeed Maintenance Adapter Instructions for Use


Intended Use: Use when maintaining Midwest® Straight Attachments and Midwest® U-Style Adapters on an automated handpiece maintenance system.

Contraindications: Do not use Midwest® Shorty® Low Speed Motors and Midwest® Contra or Right Angle sheaths with the Maintenance adapter. Do not sterilize maintenance adapter.


Warning: This adapter is for maintenance use only. Not for use on dental delivery systems.

To connect Midwest Lowspeed Straight Attachment, Contra Angle Sheath and U-style adapter to Midwest Lowspeed Maintenance Adapter:

- 1 **Align attachment with Maintenance Adapter.**

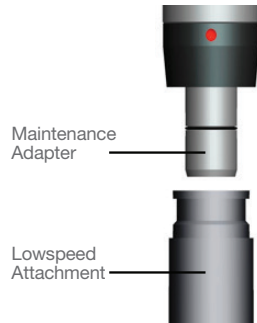


- 2 **Slide attachment onto Maintenance Adapter until it clicks into place.**

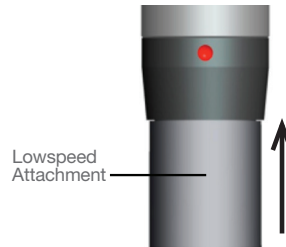

To Disconnect:

- 1 **Depress red indicator on black ring and slide attachment off the Maintenance Adapter.**

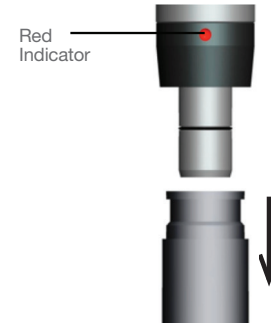

DENTSPLY
87473-0813

Manufactured for:
DENTSPLY Professional
A division of DENTSPLY International
901 West Oakton Street
Des Plaines, IL 60018-1843 USA

Imported & Distributed by:
DENTSPLY Canada
161 Vinyl Court
Woodbridge, Ontario
L4L 4A3

Technical Customer Service
1.800.800.7202