

## MANUAL MAINTENANCE

For detailed instructions refer to respective Midwest Handpiece Maintenance Instructions for Use.

### After each use of handpiece & before sterilization:

- Scrub and rinse handpiece under warm tap water.
- Remove protective cap from spray can.
- Attach correct nozzle to can.
- Hold can in upright position.
- Fully insert nozzle to backend of handpiece.
- Cover head of handpiece with Midwest® Spray Guard (REF 380150).
- Spray for one second. If excess debris appears, spray again for one second.
- Once a week spray into chuck for one second.
- If needed, connect handpiece to respective maintenance coupler.
- Expel on Midwest® Air Station (REF 380800) for a minimum of 30 seconds or till turbine reaches normal speed, and excess lubricant is expelled.
- Wipe off any excess lubricant on outside of handpiece with dry gauze or tissue.
- Bag and sterilize.
- Do not lubricate after sterilization.

### Nozzles

- Midwest® Stylus™ ATC REF 880080
- Midwest® Stylus™ and Midwest® XGT REF 380085
- Midwest® Fixed Backend and Attachments REF 380086
- Midwest® eStylus™ REF 380087

### Maintenance Couplers

- Midwest® Stylus™ ATC REF 880090
- Midwest® Stylus™ and Midwest® XGT REF 380090
- Midwest® eStylus™ REF 380088



Consult  
Instructions  
for Use



Temperature  
Limitations



Keep Out  
of Sunlight



Flammable



D67238008013

For Dental Use Only.

Volume: 500ml.



Contains: Paraffin based neutral lubricating oil, Propane, Butane.

No CFCs.

EXP

Expiry Date and LOT Number on can bottom.



Manufactured in Switzerland for:  
DENTSPLY Professional  
A Division of DENTSPLY International  
901 West Oakton Street  
Des Plaines, IL 60018-1843, USA  
800-989-8825  
www.dentsply.com

Imported & Distributed by:  
DENTSPLY Canada  
161 Vinyl Court  
Woodbridge, ON L4L 4A3, CANADA  
800-263-1437  
www.dentsply.ca

**DENTSPLY**  
PROFESSIONAL

Code No. 82620-1012

# MIDWEST® Plus Aerosol Spray

MIDWEST®

REF 380080

## INDICATIONS FOR USE

Cleaner & Lubricant for Automated and Manual Maintenance of Midwest Handpieces.

## CONTRAINDICATIONS

Do not use on Midwest® Shorty® motors.

## WARNINGS

- Pressurized container. Do not expose to temperatures exceeding 122°F (50°C).
- Do not pierce or burn, even after use.
- Keep out of the reach of children.
- Do not allow to enter sewers, surface or ground water.
- Do not dispose in household garbage.

## PRECAUTIONS

- Use only in accordance with MSDS information. For a copy call 800-989-8825.
- Do not ingest, inhale, or get in eyes.
- Use proper barrier protection while using this product. Keep unprotected persons away.
- Prolonged exposure to fumes may affect respiration.
- Use only in well-ventilated areas.
- Keep away from sources of ignition - No smoking.
- Take precautionary measures against static discharges.
- Do not spray on a naked flame or any incandescent material.

## ADVERSE REACTIONS

- If you experience breathing difficulty, supply fresh air and seek immediate medical attention.
- The product generally does not irritate skin, but if irritation occurs seek medical attention.
- If eye contact occurs, rinse opened eye for several minutes under running water. If problems persists, seek medical attention.
- If swallowing occurs, seek immediate medical attention.

## INSTRUCTIONS FOR USE AUTOMATED MAINTENANCE

For detailed instructions refer to Midwest® Automate™ (REF 380400) Instructions for Use.

- Remove protective cap from spray can.
- Open spray can compartment door.
- Insert spray can in holder clips insuring the can is fully seated against physical stops at both the top and bottom.
- Close spray can compartment door firmly and completely.
- Do not operate Midwest Automate with spray can compartment door open.